



Particulate Mask

BN-A96

USE INSTRUCTIONS:



The mask must be worn at all times of exposure and be disposed of after one time use.

1. Please read the instructions provided in this manual carefully. Any deviation from the instructions during times of exposure can reduce the overall effectiveness of the mask which may result in sickness or death.
2. Before using the mask in an occupational environment, a written mask protection program must be put into effect which meets all requirements outlined in OSHA 29 CFR 1910.134 including information on how to perform a proper seal check.
3. 3. Leave the contaminated area immediately if you experience any sign of dizziness, irritation, or other issues.
4. When not in use, store the masks away from contaminated areas.
5. Dispose of used masks after one use. Disposal of the mask must be in accordance with all applicable regulations.

SPECIFICATIONS:

- Mask Type: Standard with Duckbill Folding Style with Headband Attachment
- Storage Temperature: 26°F ~ 104°F
- Color: Blue/White

USE LIMITATIONS:

1. Do **NOT** enter a contaminated area under the following conditions:
 - The area contains less than 19.5% oxygen.
 - The area contains oil aerosols.
 - The presence of gases or vapors.
 - The contaminants in the area are unknown and can impact health or cause death.
2. Do **NOT** attempt to modify or use the mask in ways other than instructed.
3. Do **NOT** use the mask with beards or facial hair as it may prevent a good facial seal.



WARNING The mask helps protect against certain contaminants. It may not eliminate the risk of contracting disease or infection. Misuse may result in sickness or death. Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals.

FITTING INSTRUCTIONS:



STEP 1

Unfold the mask and gently bend the nosepiece to form a modest curve.



STEP 2

Remove both headbands from the unfolded mask.



STEP 3

Cup the mask under your chin with the nosepiece facing upwards.



STEP 4

Pull both headbands over your head and rest them on your neck, under your ears.



STEP 5

Pull the top headband up over your ears and rest it on the crown of your head.



STEP 6

Firmly press down the nosepiece to conform to the shape of your nose. Adjust and secure all edges and then perform a seal check.

PERFORMING A SEAL CHECK:

It is necessary to perform a seal check on the mask every time you wear it.

Cup both hands around the mask and inhale and exhale forcefully several times. If you notice any air leakage, adjust the nosepiece and headbands until the leakage is corrected. Pull the headbands back, up, or, if necessary, shorten them by knotting. Once corrected, repeat the seal check again.

⚠ WARNING The mask should not be used without a proper fit.

Learn more at **BNX.com**



Manufactured by
BNX Converting, LLC.
16727 Park Row Houston, TX 77084 USA

