

AccuMed

Instruction Manual

ANC-201 Baby Nasal Aspirator



V.1
October 27, 2017
www.AccuMed.com

INTRODUCTION:

The AccuMed Nasal Aspirator is the latest development in baby care products that offers efficient, hygienic, and a safe solution to suction out nasal mucus. We hope this product is useful to you. We recommend reading this user manual in its entirety for instructions to clean your child's nose, best use of product and maintenance.

Children can experience mucus build-up leading to nasal passage blockage which can range from mild discomfort to a life-threatening condition. Newborns are obligate nasal breathers, meaning they must physiologically breathe through their nose as opposed to their mouth. As babies grow, most will still prefer nasal breathing. Therefore, nasal mucus must be frequently removed to facilitate feeding and avoid infections that can spread to sinuses, ears and the general respiratory tract.

The AccuMed Nasal Aspirator removes mucus in seconds, relieving your baby's discomfort and allowing your baby to breathe easy.

Please keep this instruction manual for reference.

UNPACKING CHECK:

Please open the package and check that all items are included:

- AccuMed Nasal Aspirator
- 3 Silicone Tips
- 2 AA Batteries
- 25 Disposable Hygienic Caps
- 2 Extra Silicone Joints
- Instruction Manual

Examine product and all accessories to check that nothing damaged. Perform installation and operation following this user manual. If any damage or operation problem is observed, please contact AccuMed:

To contact AccuMed support, please do one of the following:

- Visit <http://accumed.com/contacts/> and use the email form
- Call AccuMed Support at 1(713) 904-1955
- Email AccuMed Customer Support support@accumed.com

SAFETY INSTRUCTION:

1. Device should ONLY be used to remove nose secretions and mucus in newborns, babies, and toddlers. Do not use this product for any other body part other than the nasal cavities or for any other purpose;
2. Only use accessories provided in our package;
3. Functional damage may occur if unit is dropped or thrown;
4. Keep device out of the reach of children;
5. Do not use rechargeable batteries or try to recharge batteries included;

6. Remove batteries when product is not expected to be used for an extended period of time;
7. Follow battery installation instructions provided in this manual;
8. Do not disassemble or attempt to repair product.
9. Do not submerge device in liquid or expose to direct sunlight for a long period.

BEFORE INITIAL USE:

1. Cold sterilize all accessories before and after each use especially probes.
2. Wash hands before using the accessories.
3. Ensure waterproof seal is intact each time.



RECOMMENDATIONS FOR PARENTS:

1. Wash hands before using product.
2. Keep baby's head upright during operation and to allow air to flow through.
3. Soften nasal secretion with a few drops of saline solution before operation.
4. Make sure the baby's mouth is open during operation so air could drift through nostril.
5. To avoid infections:
 - a) Do not kiss sick or suspected sick babies;
 - b) Do not kiss a baby when you are sick;
 - c) Keep sick babies from contacting other children or adults;
 - d) Do not cross use baby's personal items with others.

PRODUCT FEATURES:

- 3 options setting for aspiration
- Silicone tip sizes for ages:
 - 3 months and under
 - 3 months to 1 year
 - 1 year and above

PRODUCT DESCRIPTION:

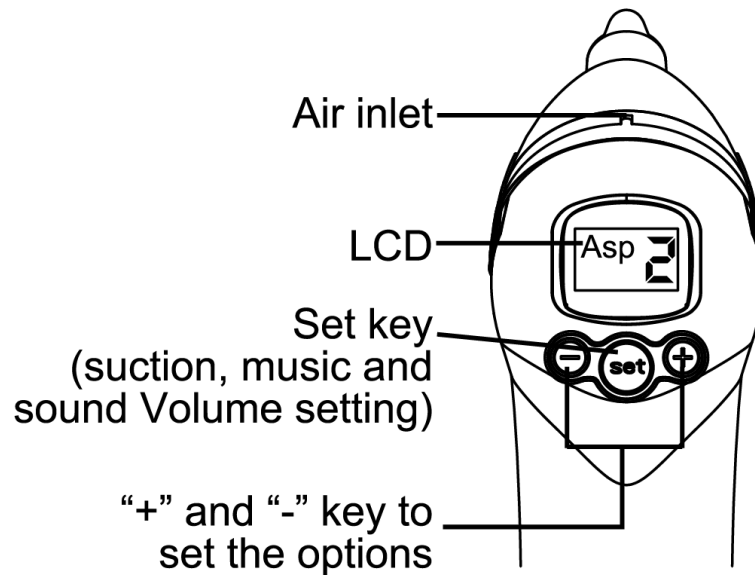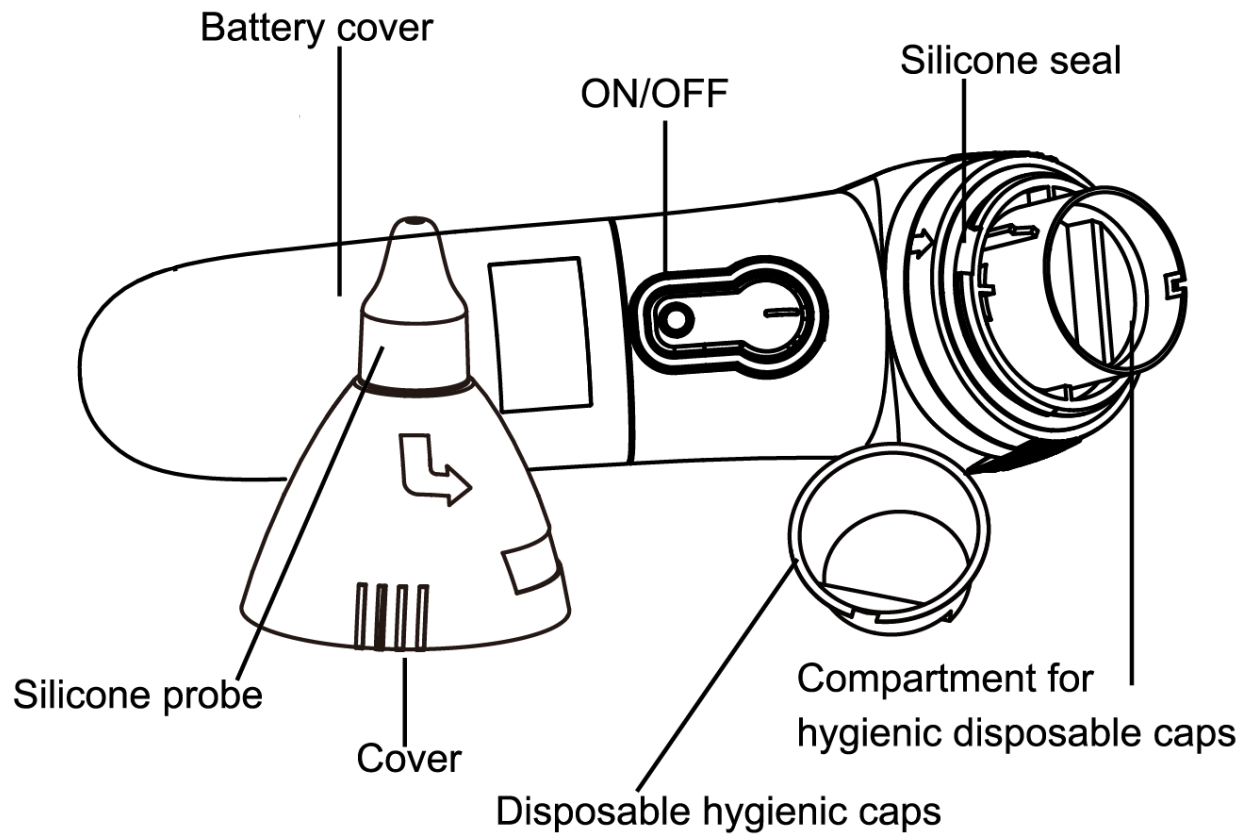

TECHNICAL FEATURES FOR ACCUMED BABY NASAL ASPIRATOR:


Maximum aspiration power at 100%: 50 kPa.

Menu		0	1	2	3
ASP	Aspiration	-	30% SOFT	70% MEDIUM	100% HIGH

Default: ASP: 2

When switched to On, the device defaults to Aspiration Level Medium (Asp 2).

BATTERY INSTALLATION:

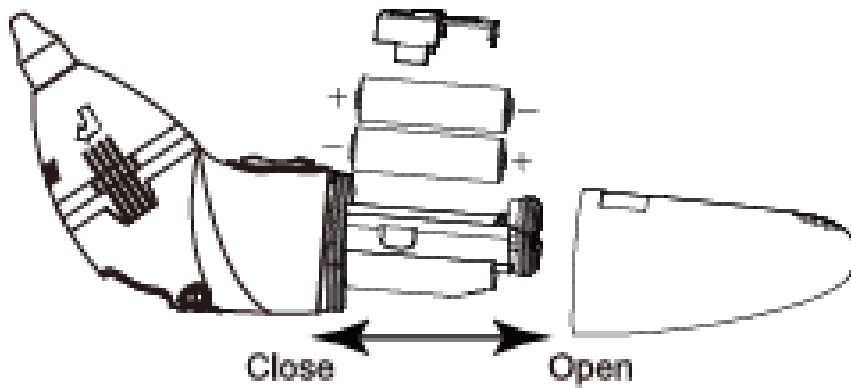


1. Pull down the battery cover to open the battery compartment;
2. Remove the battery cap from the top;
3. Check the polarity position and insert the two AA batteries in place;
4. Place battery cap and attached battery cover back to body of device.

NOTE: Do not use rechargeable batteries. Do not pair a new battery with an old one;

UNIT ASSEMBLY:

- 1) After inserting batteries and reattaching the battery cover ①,
- 2) Lightly twist cover ⑤ to remove.
- 3) Place a disposable hygienic cap in the disposable cap compartment ④.
- 4) Ensure silicone seal is properly positioned to avoid any spills while in use ②.
- 5) Position cover ⑤ in correct place and twist to attach to the rest of the unit.
- 6) Choose the appropriate silicone tip size and place on probe ⑥
 - Pink color are for the newborns to three months of age;
 - Blue color are for three months to twelve months of age;
 - Orange color are for above twelve months of age.

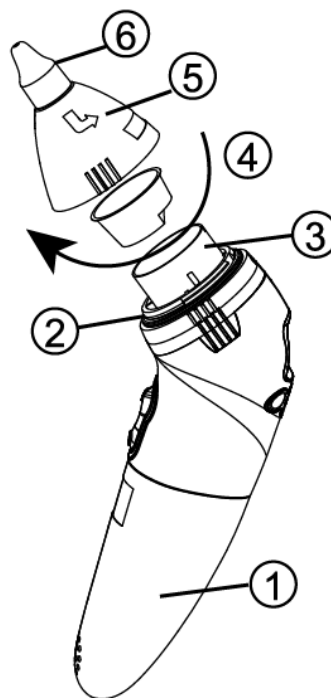

- ① Battery Cover
- ② Silicone Seal
- ③ Sealing Base
- ④ Disposable Cap Compartment
- ⑤ Cover
- ⑥ Silicone probe

INSTRUCTIONS FOR USE:

- 1) Turn on the device by pressing the button "I".
- 2) Press and hold the "SET" key for three seconds to adjust the aspirator level.
- 3) The screen will flash "ASP". "+" and "-" keys are for the aspiration strength (1 - soft, 2- medium, 3- high).



NOTE: The Aspirator is level 2 by default.

- Make sure baby's mouth is open during operation to so that air drifts through the nostril cavity.
- When using saline solution, make sure nasal secretion is sucked away.
- Insert probe gently, keeping probe tip straight and away from nostril wall.
- Insert probe slowly, avoiding any harsh or sudden movement.
- Change the hygienic cup when it is necessary.

MAINTENANCE:

- 1) Open top cover and clean device with detergent.
- 2) Sterilize removeable parts with boiling water or steam.

- 3) Clean device and parts with a dry and soft cloth after sterilizing.
- 4) Do not expose to direct sunlight or damp area.
- 5) Do not use erosive chemicals to clean device.

To contact AccuMed support, please do one of the following:

- Visit <http://accumed.com/contacts/> and use the email form
- Call AccuMed Support at 1(713) 904-1955
- Email AccuMed Customer Support support@accumed.com

For more information please visit www.AccuMed.com

Manufactured for:

AccuMed

16727 Park Row Drive, Houston, TX 77084

(713) 904-1955