

## User Information

Model number: B702/ B702V/ B704/ B704V (FFP2 NR)

### Important:

The filtering half masks have been designed to meet the requirements of REGULATION (EU) 2016/425 and harmonies standard EN149:2001+A1:2009.

This single use respirators are designed to protect against solid and liquid aerosols. The filtering half masks are classified as FFP2 NR.

Before using these masks reading the user information and check the classification to which respirator belongs on the packaging.

### Applications:

FFP2 NR filtering half masks are intended use to protect the user against non-toxic solid and liquid aerosols, as according filtering efficiency (Max. 8%) and their maximum total inward leakage (Max. 11%)

### Fitting Instructions:

- Hold the respirator in hand with the nosepiece at your fingertips. Allow the headbands hang freely below hand.
- Cup the respirator firmly against your face with the nosepiece on the bridge of your nose.
- Stretch and pull the lower headband over the head and position below your ears. Stretch and pull the top headband on the back of your head above your ears. Press soft nosepiece to confirm snugly around the nose.
- To test fit for respirator without exhalation valve, cup both hands over the respirator and exhale vigorously.
- To test fit for respirator with exhalation valve; cup both hands over the respirator and inhale sharply.

A negative pressure should be felt inside of respirator if air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fitting.

- Change respirator immediately if breathing becomes difficult or respirator become damaged or distorted or a proper face fit can not be maintained. Careful observance of these instructions is an important step in safe respirator use.

### Warning:

1. Failure to following all instructions and limitations on the use of the product could reduce the effectiveness of respirator and result in sickness or death.
2. A properly selected respirator is essential to protect your health, before using this respirator, consult an industrial hygienist or occupational safety professional to determine the suit ability

for your intended use.

3. This product does not supply oxygen, use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%
4. Do not use when concentrations of contaminants are immediately dangerous to health or life.
5. Do not use in explosive atmospheres.
6. Leave work area immediately and return to fresh air if breathing become difficult or dizziness or other distress occurs.
7. Facial hairs or bears and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter or modify this respirator in any way.
9. Only for single shift use. No maintenance necessary. Discard this respirator after single shift use, the respirator shall not be used for more the one shift.
10. Keep respirator in the original box, away from direct sunlight until use.
11. Respirator should be stored and transported in its original box, in a clean, cool and dry atmosphere to ensure that is protected against dust, chemical, moisture, excessive heat, direct sunlight or accidental damage.
12. Before using respirator, check for visible damage and expiration date, damage mask or expired respirator may not be used.
13. The period of obsolescence of the PPE marked on packaging in Month and Year.

**Meaning of marking:**



B702 FFP2 NR  
EN149:2001+A1:2009

**CE0194**



- Trade mark

EN149:2001+A1:2009 - European harmonies standard

B702 – Model number

FFP2 - Classification

NR - The respirator shall not be used for more the one shift

**CE** -Conform to PPE regulation; Height: 5mm

0194 - Notified body number



Verison:201905

EU declaration of conformity can be found in <https://www.beibeisafety.com>

Notified body:

Inspec International Ltd (0194)

56 Leslie Hough Way

Salford Greater Manchester

M6 6AJ.

UK

Manufacture

BeiBei Safety Co., Ltd

No. 16, Lu Kong South 5<sup>th</sup>.

Lu Kang Town, Changhua Hsien

Taiwan

[www.beibeisafety.com](http://www.beibeisafety.com)